


BENEFITS/AVANTAGES/VORTEILE/VOORDELEN

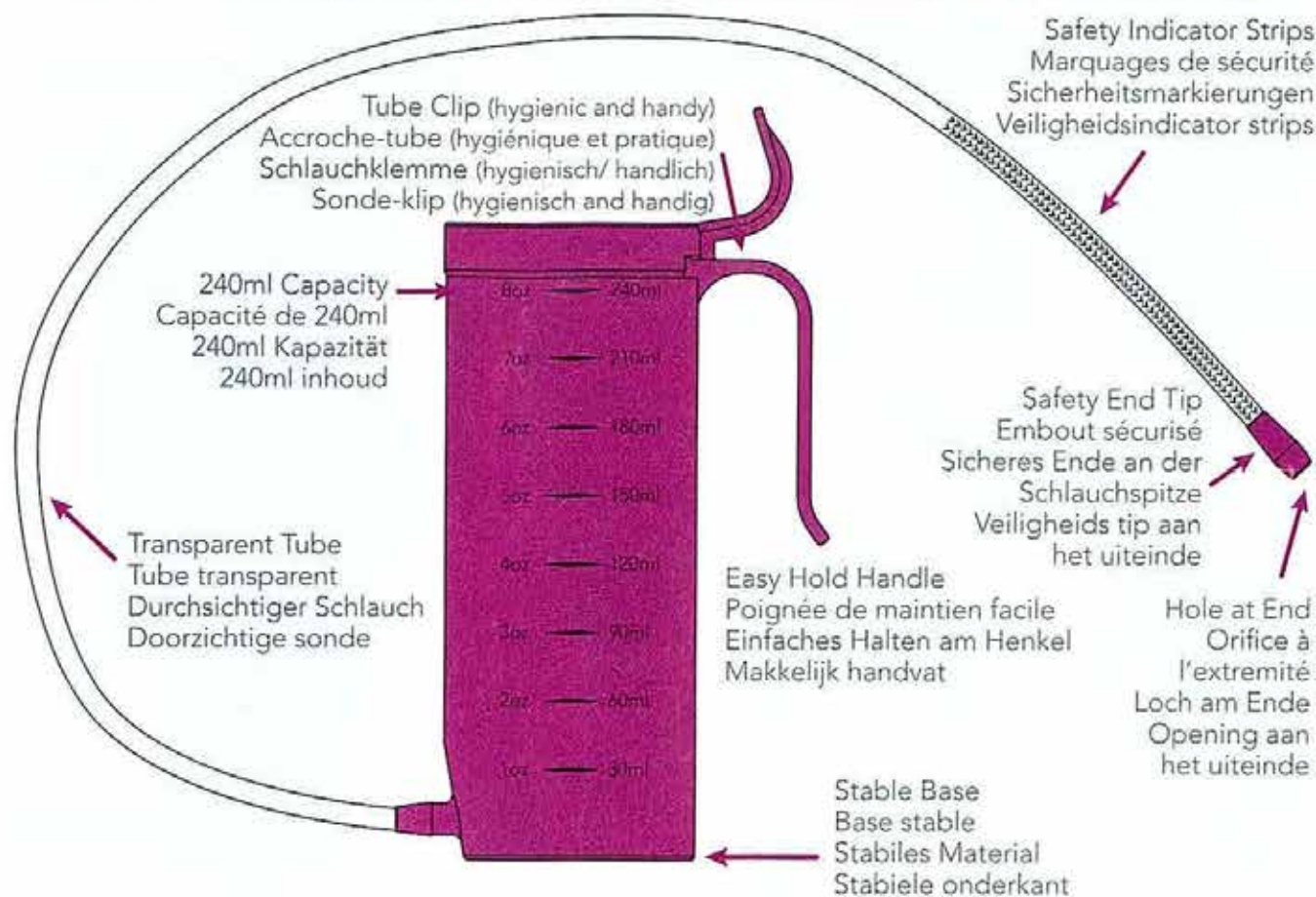

How to Use:

- 1) Gently pass the tube into the oesophagus, until the safety indicators are within the mouth.
- 2) Raise vial. Ensure the head and neck remain above the stomach and ensure the tube stays in position.

Tips for safe use of your Trusti® Tuber LK:

Hold the vial with middle finger in the handle above the tube clip and two fingers in the handle below. This way you can easily hold and guide the tube with thumb and index finger. Even better this can be done with a prefilled vial because the tube is held above the fluid level. Making one-person operation easy!

To ensure the tube stays in position during fluid flow – secure the tube with the same hand holding the animals head.

Ensure the head and neck of the lamb or kid goat stays above stomach level.

The tube safety marks indicate when you have passed the tube a safe distance past the airway entrance (throat) in a lamb or goat kid up to 6kg. When the printing cannot be seen outside the animals mouth you are far enough. Consider this distance a minimum as it is safer to pass the tube further than not far enough. Aim for the lower neck or further.

You can check tube position in the oesophagus by looking in the groove on the left side of the animal's neck. The oesophagus is a soft collapsed tube enabling you to see the tube tip moving. Use the tube clip to keep the tube hygienic and above the fluid line when resting or filling your Tuber.

In very cold temperatures you may like to warm the tube to soften.

Always pass the tube gently.

To stop flow during feeding simply lower the vial.

Wait until all liquid has run through before gently removing the tube.

We recommend washing your Trusti® Tuber LK soon after use with mild disinfectant and warm water. Do not soak in acid.

Please watch the demonstration videos at www.ANTAHI.com

Disclaimer: Antahi Innovations will not be liable for any loss or harm caused to any animal, any indirect, special or consequential loss or damage (whether for loss of profit or otherwise), costs, expenses or other claims arising out of or in connection with the use of this product.

FURTHER ADVANTAGES

- One handed operation
- Easy clean design and material
- Milk into vial straight from the ewe/nanny
- Clean the whole tube – no build up in the tip
- Gravity fed, not pressurised

AUTRES AVANTAGES

- Utilisable d'une seule main
- Conception et matériaux optimisés pour un nettoyage facile
- Remplir le flacon directement du lait de la brebis
- Nettoyage facile du tube, pas d'amas dans l'embout

WEITERE VORTEILE

- Einhändig bedienbar
- Marp en materiaal
- Leicht zu reinigende Konstruktion und Material
- Milch gelangt direkt vom Mutterschaf bzw. Mutterziege in die Flasche
- Den ganzen Schlauch säubern – alle Reste aus der Spitze entfernen

VERDERE VOORDELEN

- Met een hand te gebruiken
- Makkelijk schoon te maken ontwerp en materiaal
- Melk rechtstreeks in de fles vanuit de ooi/geit
- Hele sonde makkelijk schoon te maken, ook het uiteinde